4140 // SAFE AIR

Atemschutzgeräte / Risikokategorie III Respiratory protective devices / Risk category III

ENSURESAFETY

DE

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EN 149:2001 + A1:2009

Respiratory protective devices

Filtering half masks to protect against particles (solid and liquid aerosols), except for escape purposes.

Class: FFP3 NR D

Instructions for wearing the product: Preform the nose clip so that it corresponds approximately to your nose shape. Then hold the mask over your mouth and nose and place the fastening straps around your head. The lower fastening band should run around the neck. The upper fastening band should run above the ears around the back of the head. Now adjust the nose clip to your nose and make sure that the mask fits tightly. For example, you can check a firm and tight fit by wrapping the mask with both hands, inhaling deeply and exhaling vigorously. If air escapes from the nose bow, it must be pressed more strongly. If other areas are

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leaking, the fit of the mask must be adjusted. Replace the mask immediately if breathing becomes more difficult, or if the mask is damaged or deformed and therefore an optimal seal is no longer guaranteed.

Failure to follow these instructions may reduce the effectiveness of the mask and can result in serious consequences. FFP1: Against CMR substances, radioactive substances and airborne biological agents classified in risk groups 2 and 3 and enzymes. Protects up to four times VdGW (multiple of the limit value).

FFP2: Against CMR substances and radioactive substances as well as airborne biological agents classified in risk group 3 and enzymes only after risk assessment. Protects up to ten times VdGW (multiple of the limit value).

FFP3: No comments or restrictions. Protects up to thirty times VdGW (multiple of the limit value).

NR: Not reusable.

D: Dolomite clogging test passed

Warnings: This product may not be used for more than one shift. The fit of the particle-filtering half masks must be checked before use. It is unlikely that the leakage requirements will be met if facial hair is under the seal line.

The protective function of filtering half masks also depends on the ambient atmosphere. Use only in adequately ventilated environments. Do not use if the atmosphere contains less than 17% oxygen. Filtering half masks must not be used if the concentration of pollutants is a direct threat to health or life. Do not use in explosive atmospheres. Leave the environment immediately if breathing becomes difficult, dizziness or exhaustion occurs. No changes or modifications may be made to the mask.

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