

TECHNICAL DATA SHEET

52995 // PROTECTASEPT FX PLUS

Type:	Surface disinfection, neutral
Content:	5,000 ml
Packaging Unit:	1 piece



Description

Ready-to-use alcohol-free rapid disinfection for disinfecting surfaces of all kinds, fragrance: neutral, alcohol-, aldehyde-, surfactant- and phenol-free, VAH-compliant tested, Dermatest: very good, no hazardous material / substance, not subject to labelling, non-flammable, fully virucidal, 100% biodegradable, comprehensive spectrum of action within 1 minute

General information

Hygiene is used throughout (medical) operations to prevent infections in patients and staff. In order to be able to guarantee high medical standards, careful and thorough hygiene in practices, hospitals, etc. is essential. The application of cleaning agents and disinfectants according to a fixed time schedule is laid down by law. The aim is to eliminate infection-relevant germs with standardised, quantifiable proof of effectiveness to prevent nosocomial infections. The surface disinfectants are used both for routine disinfection as „prophylactic disinfection“ and for targeted disinfection of visibly contaminated surfaces or surfaces or objects that could be contaminated. From the point of view of work organisation, exposure times play an important role. Surfaces that could become sources of infection in the case of rapid successive interventions on patients should be disinfected with fast-acting procedures.

Application areas

Ready-to-use spray and wipe disinfectant with the novel, water-based active ingredient NEUTRAL+ION®. Free of alcohol, Aldehydes, surfactants and phenols and particularly mild.

Application

The ready-to-use rapid disinfection is applied to surfaces of all kinds from a distance of approx. 30 cm. Alternatively, the solution can also be applied directly to a lint-free, undyed disposable cloth. Make sure that the surfaces to be disinfected are completely wetted. Then allow to act briefly and, if necessary, remove excess preparation with a lint-free, undyed disposable cloth. Check material compatibility in a concealed area.

Composition

In 100 ml contain: > 0.12 g sodium hypochlorite

Exposure times

Bactericida EN13727:2015 (5 Min.)
EN13697:2015 (5 Min.)

Levurocide: EN13697:2015 (5 Min.)

Mycrobactericide: EN14348:2005 (5 Min.)

Sporicide: FprEN17126:2018 (1 Min.)

Fungicide: EN13624:2013 (15 Min.)

Virucidal: EN14476:2013 (1 Min.) prEN16777:2016 (15 Min.)

Safety instructions

Use biocide safely and carefully. Read the enclosed safety data sheet before use! Store at room temperature. Protect from direct sunlight.

Expert report

Our products are scientifically examined and tested according to the latest methods.

Gemeldetes Biozid-Produkt

Trade name: Nitras Protectasept Neutralion

Registration number: N-75621

Notification date: 21.02.2018

Maximum marketability (ChemBiozidMeldeV): 31.12.2024

Article number: 52995